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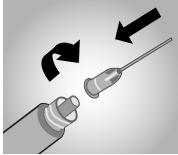
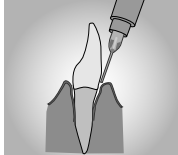
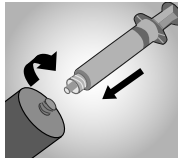
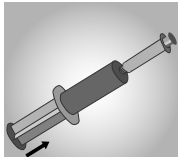



**Refill-Container 02557**  
50 ml Spritze

**Spülkanülen 02629**  
25 St.

## Mouth rinsing solution, contains chlorhexidine

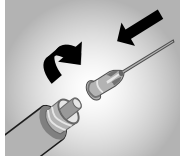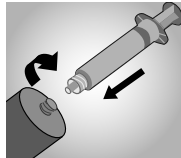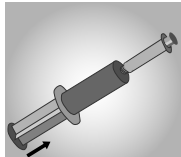**Fig. 1****Fig. 2****Fig. 3****Fig. 4**

### 1. Preparation

Remove the cap from the syringe by twisting anticlockwise. Attach the blue rinsing cannula and lock by twisting clockwise (Fig. 1). If necessary, the rinsing cannula can be bent slightly, but do not snap. The outlet area of the rinsing cannula is blunt and must not be allowed to become misshapen.

### 2. Application

Hold the syringe with the rinsing cannula uppermost and depress briefly until liquid emerges from the cannula opening. Insert the cannula into the affected area and rinse by applying gentle pressure (Fig. 2). Do not apply excessive pressure to prevent injuring. Where necessary, any rinsing fluid that escapes can be aspirated. Allow to act for at least 60 seconds, then spit out and if necessary rinse the oral cavity with water. Seal the syringe with the original cap for storage.

### 3. Refilling

Remove rinsing cannula from syringe. The syringe piston must be completely depressed. Hold the refill container always vertically with its opening right up. Remove cap from refill container. After the cap and the rinsing cannula have been removed, the syringe and the refill container can be connected. Place the syringe with the opening on the container and join the two by screwing together (Fig. 3). The syringe is now filled by carefully depressing the plunger of the container (Fig. 4). Exert only sufficient pressure to fill the syringe. If too much pressure is applied, it may be possible to extract the plunger. After filling, disconnect the syringe and reseal the container tightly.

### Important hints

- Do not swallow rinsing solution.
- For greater comfort, it is recommended that the rinsing solution should be warmed by holding the syringe under warm running water.
- Temporary changes in taste may occur.
- **smartclean** is alcohol-free and does not contain sugar.

### Indications for use:

- Rinsing after professional cleaning or pre-treatment

### Technical Data:

- **Volume:** 5 ml syringe
- **Colour:** orange-transparent
- **Reaction time:** approx. 60 sec.
- **Application:** recommended at 37 °C / 99 °F (body temperature)

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